Lifetime Limited Warranty

Your Guardian® product is warranted to be free of defects in materials and workmanship. This warranty is limited to the original consumer purchaser.

This warranty does not cover product failure due to owner misuse or negligence, or normal wear and tear. The warranty does not extend to non-durable components, such as but not limited to rubber accessories and grips which are subject to normal wear and tear and replacement. This warranty gives you specific legal rights, and you may have other rights, which vary from state to state.

In the event of a defect covered by this warranty, we will at our option repair or replace the product. For warranty service please contact the retailer where the product was purchased. If you have a question about your Guardian® brand product or this warranty, please call us toll-free at: 1-800-MEDLINE (633-5463).

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GUARDIAN

Medline's mission is to improve people's lives by creating innovative, high quality products that promote independent, active lifestyles. Visit www.medline.com for other products that can enhance your life.

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Accessories to enhance the flexibility of your walker.

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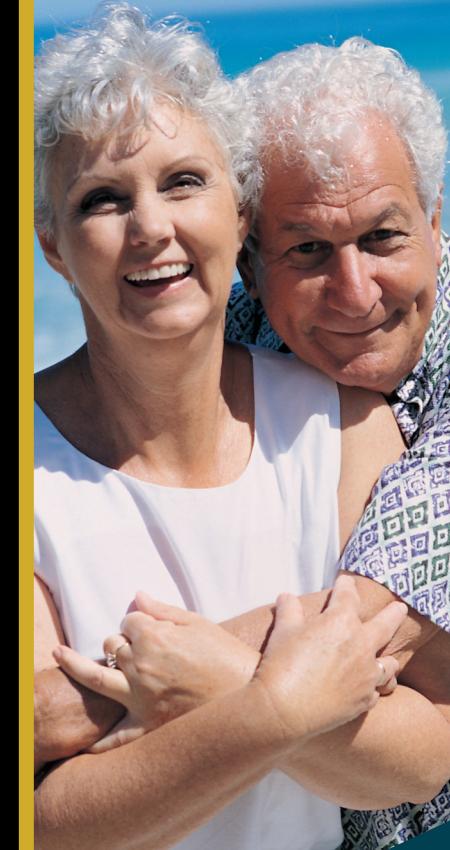
Figure 4

Figure 5

! WARNING!
Failure to follow these instructions may result in injury.

- Do not use your walker until the handles are properly adjusted and securely tightened.
- Do not sit on walker if ground is sloped or uneven.
- Do not exceed the maximum weight capacity listed on the walker.
- Do not use this walker as a wheelchair; doing so may result in a tip-over, resulting in injury.

I. ASSEMBLY

A. INSTALL HANDLE BARS

1. Remove adjustment knob and bolt from both handles.
2. Insert handle into walker frame (Figure 1).
3. With your arm relaxed at your side, position the handle so the grip is level with the bend in your wrist.
4. Insert the bolt through the plastic washer, frame and handle (Figure 2).
5. Attach knob securely.

B. INSTALL BACKREST

1. Insert backrest into frame (Figure 3) until the snap buttons engage. Ensure backrest is securely fastened.

C. INSTALL BASKET

1. Attach basket hooks to walker frame (Figure 4).

II. USING YOUR WALKER

A. FOLDING YOUR WALKER

1. Pull up on the seat.
2. To unfold, place all wheels on the ground. With one hand, push down on the seat.

B. LOCKING AND UNLOCKING YOUR WALKER'S BRAKES

1. To lock brakes, push down on brake loop handle until it "clicks" into the locked position (Figure 5).
2. To unlock, pull up on brake loop handle.

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