230

Universal Procedures Table

Ritter®
by MIDMARK

[English](#)

[Español](#)

[Français](#)

User's Guide

Product Information

(The information below is required when calling for service)

Dealer [name / phone]:

Date of Purchase:

Model / Serial Number:

Midmark Authorized Service Company [name / phone]:

**Model / Serial Number
Location**

MA684200p

Table of Contents

Important Information

<u>Safety Symbols</u>	4
<u>Intended Use</u>	5
<u>Electrical Requirements</u>	5
<u>Electromagnetic Interference</u>	5
<u>Disposal of Equipment</u>	5
<u>Transportation / Storage Conditions</u>	5
<u>Safety Instructions</u>	6
<u>Foot Control Connection</u>	6

Operation

<u>Power To The Table</u>	7
<u>Base Up / Down</u>	8
<u>Back Up / Down</u>	9
<u>Foot Up / Down</u>	10
<u>Tilt Up / Down</u>	11
<u>Paper Roll</u>	12
<u>Stirrups</u>	13
<u>Treatment Pan</u>	14
<u>Duplex Receptacle (option)</u>	15
<u>Headrest Positioning</u>	16
<u>Knee / Chest Position</u>	17
<u>Rotational Base (optional)</u>	18

Maintenance

<u>Cleaning</u>	19
<u>Preventative Maintenance</u>	19
<u>Calling for Service</u>	19

Troubleshooting

<u>No Foot Up Function</u>	20
----------------------------------	----

Specifications

<u>Weights, Dimensions, Electrical Specifications</u>	21
<u>Model Identification / Compliance Chart</u>	22
<u>Range of Motion & Dimensions</u>	23

<u>Limited Warranty</u>	25
-------------------------------	----

Important Information

Safety Symbols

WARNING

Indicates a potentially hazardous situation which **could** result in serious injury.

Caution

Indicates a potentially hazardous situation which could result in injury if not avoided.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice, or condition.

Consult User Guide for important information

Proper Shipping Orientation

Fragile

Type B, Applied Part

Protective earth ground

Keep dry

Maximum stacking height
(palletted units)

Fuse rating specification

IPX1

Protected against dripping water

Duty Cycle (motor run time):
30 seconds ON, 5 minutes OFF

Pan must be fully retracted to
enable Foot Up function.

Patient weight limit

This product has been evaluated with respect to electrical shock, fire & mechanical hazards only, in accordance with UL60601-1 and CAN/CSA C22.2 NO. 601.1.

Intended Use

This product is intended to be used as a chair / table to provide positioning and support of patients during general examinations and procedures conducted by qualified medical professionals.

Electrical Requirements

Caution

When using high frequency surgical devices or endocardial catheters:

- *Use non-conductive material to insulate patient from metal portions of the table.*
- *Consult operating instructions for the device before using in conjunction with drawer or upholstery heater.*

Failure to comply may result in electric shock or burns to patient.

Electromagnetic Interference

This Midmark table is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this table:

- Remove interfering device from room
- Plug table into an isolated circuit
- Increase separation between table and interfering device
- Contact Midmark if interference persists

Disposal of Equipment

At the end of product life, the table, accessories, and other consumable goods may become contaminated from normal use. Consult local codes and ordinances for proper disposal of equipment, accessories and other consumable goods.

Transportation / Storage Conditions

Ambient Temperature Range..... +41°F to 100°F (+5°C to +38°C)
Relative Humidity..... 10% to 90% (non-condensing)
Atmospheric Pressure 500hPa to 1060hPa (0.49atm to 1.05atm)

Safety Instructions

Caution

Be sure that all personnel and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.

Hand / Foot Control Connection

There are two hand / foot control cord inlets. One is located on the base column at the foot end of the table, and one is located on the PC board cover at the head end. These cord inlets are interchangeable.


Equipment Alert

Never plug hand / foot control into any other outlets. Doing so may result in damage to the equipment.


To connect hand / foot control cord...

- A) Slide narrow portion of cord connector into bracket.
- B) Connect cord to inlet (see Note).
- C) Tighten locking ring.

Note: The cord connector is "keyed" and must be oriented properly to connect.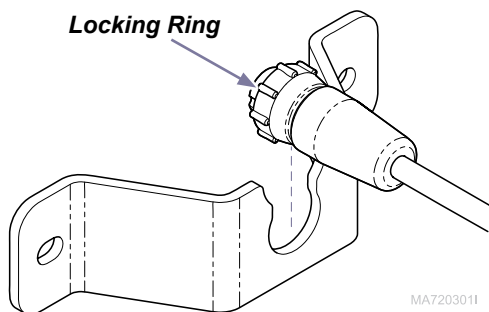

MA7203011

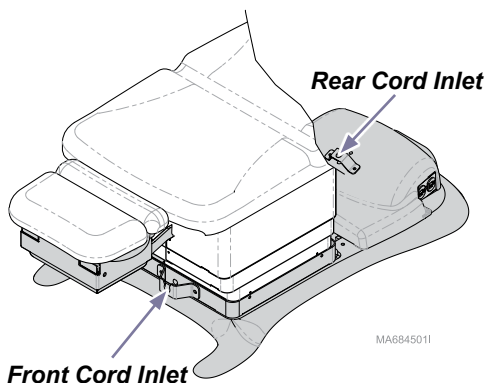MA6845011

Operation

Power To The Table

If the table has no power...

All functions will be inoperable.


Before calling for service...

- A) Check power cord connections.
- B) Check / reset facility circuit breaker.
- C) Check / replace table fuse(s).

If Power Indicator Light is...

ON: Table has power.

OFF: No power to table.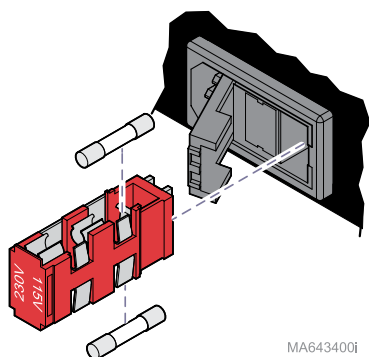

Before replacing fuses...


Disconnect power cord from table.

To replace fuses...

[Refer to Specifications section for correct fuse size & ratings]

- A) Open the fuse holder cover.
- B) Remove fuse holder & faulty fuse(s).
- C) Install new fuse(s), then replace fuse holder.

Attention: Voltage setting that shows thru window of fuse holder cover must match facility supply voltage.

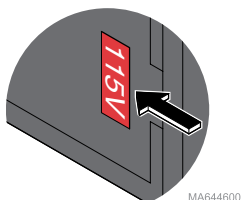Equipment Alert

Incorrect voltage setting *will* result in damage to the table.

Base Up / Down

To move the Base UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Up / Down button is released.
Maximum / minimum position is reached.

Max.
Standard base: 40 in. (101.6 cm)
Rotational base: 43.5 in. (110.5 cm)

Min.
Standard base: 22.5 in. (57 cm)
Rotational base: 26 in. (66 cm)

Back Up / Down

To move the Back section UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.

Foot Up / Down

Attention!

The Foot Up function is disabled when the treatment pan is pulled out.
(You will hear several “beeps” when Foot Up is activated with the pan pulled out.)

To resume Foot Up operation:
Push the pan all the way in.
Press Foot Up button.

To move the Foot section UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.

Max.
 $0^{\circ} \pm 0.5^{\circ}$


Min.
 $90^{\circ} \pm 1^{\circ}$

Tilt Up / Down

To Tilt the seat section UP or DOWN...
Press and hold the appropriate button.

Movement stops when...
Maximum / minimum position is reached.
Up / Down button is released.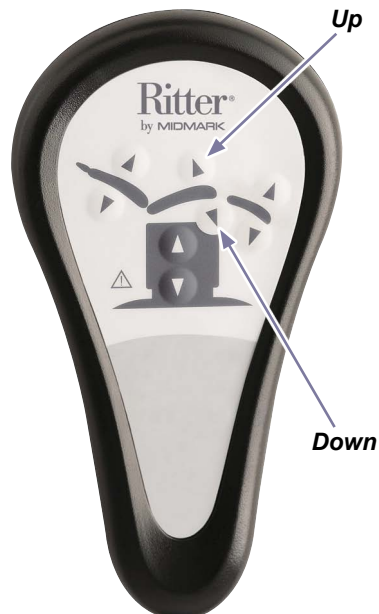


Paper Roll

*Paper Roll Size (max.)..... 18 in. long x 3.5 in. diameter
(45.7 cm x 8.9 cm)*

To install the paper roll...

- A) *Pivot paper roll cover back to open.*
- B) *Install paper roll, then pull paper over table.*
- C) *Pivot paper roll cover forward to close.*

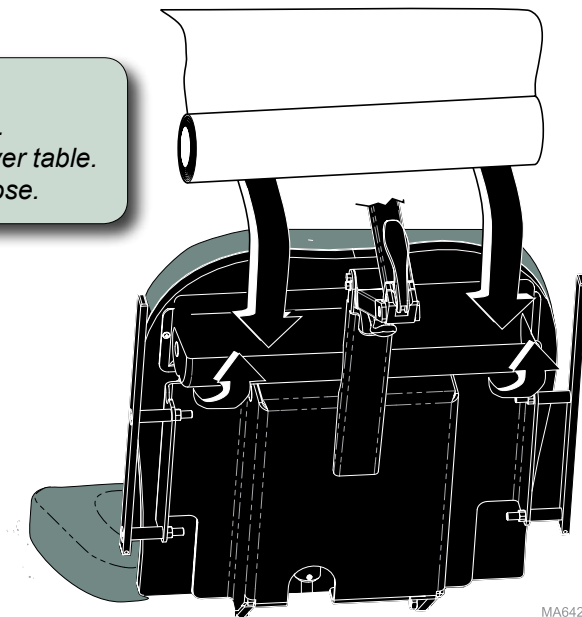MA642601I

Stirrups

Caution

Never use stirrups to lift table.

Be sure the stirrups are locked in place before using.


The stirrups will not support the patient's entire weight / excessive force.

To position the stirrups...

A) Pull the stirrup out, then unfold.

B) Lift the stirrup slightly, then move it left or right as desired.

C) Release stirrup to lock in position.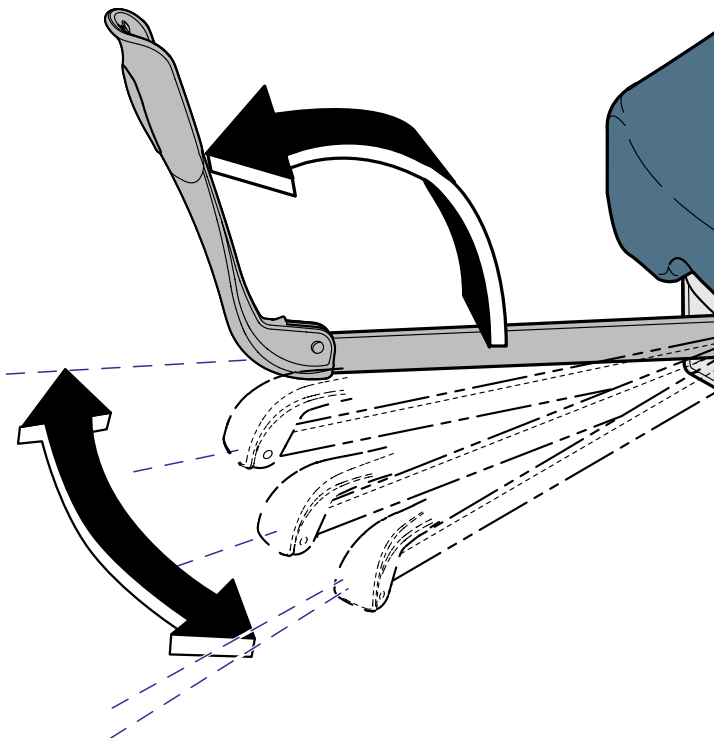

MA602500i

Treatment Pan


Attention!

The Foot Up function is disabled when the treatment pan is pulled out.
(You will hear several “beeps” if Foot Up is activated with the pan pulled out.)

*To resume Foot Up operation:
Push the pan all the way in.
Press Foot Up button.*

To access the treatment pan...

- A) Lower the foot section.***
- B) Pull the treatment pan slide out.***

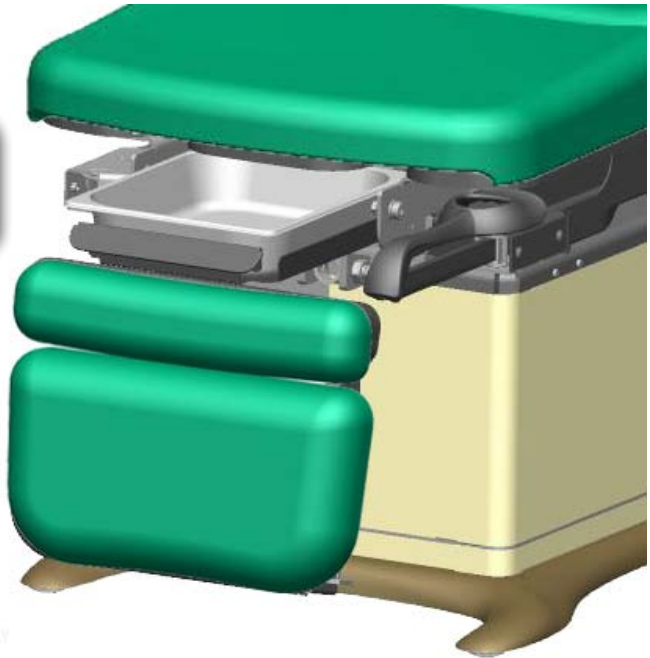Duplex Receptacle (optional - not available on 230 volt models)

The duplex receptacle provides power for accessories used during medical procedures. Power to the receptacle will be interrupted if the maximum load is exceeded.

Maximum Load [both receptacles combined] 115 VAC, 3 amps, 50/60 Hz

Equipment Alert

*Exceeding the maximum load for the receptacle may result in blown fuses. If this occurs, all table functions will be disabled until fuses are replaced. [Fuse replacement procedure is shown on the **Power To The Table** page.]*

Headrest Positioning


Caution

Be sure headrest is locked securely in position before use.

To adjust the headrest at pivot points 1 and/or 2...

- A) Pull upper handle outward to release.
- B) Move pivot point(s) to desired position.
- C) Push upper handle inward to lock in place.

[For clarity, the upholstery is not shown]

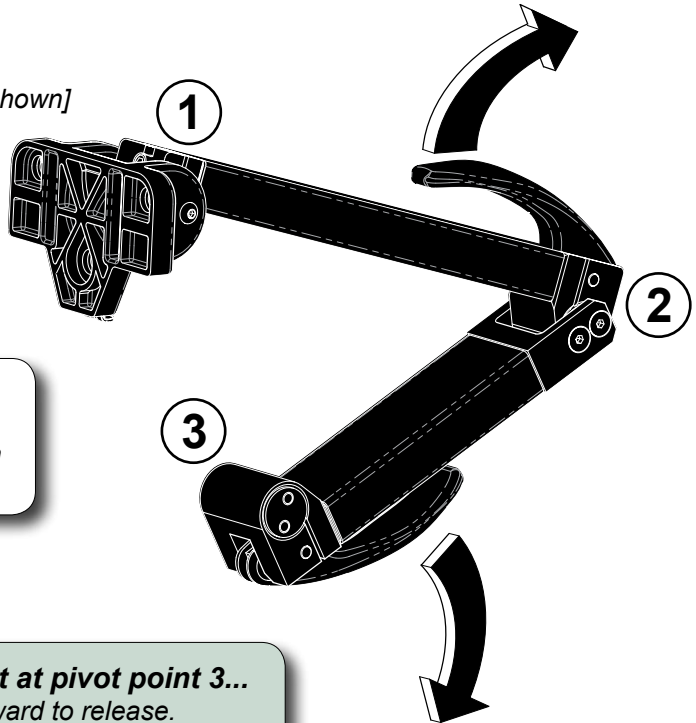Note

The handle tension is adjustable.
Contact your service provider if an
adjustment is required.

To adjust the headrest at pivot point 3...

- A) Pull lower handle outward to release.
- B) Move pivot point to desired position.
- C) Push lower handle inward to lock in place.

MA642200i

Knee / Chest Position


Caution

Be sure foot section is locked securely in position before use.

To remove the foot section...

- A) Lift release levers on either side of foot section.
- B) Pull foot section straight out.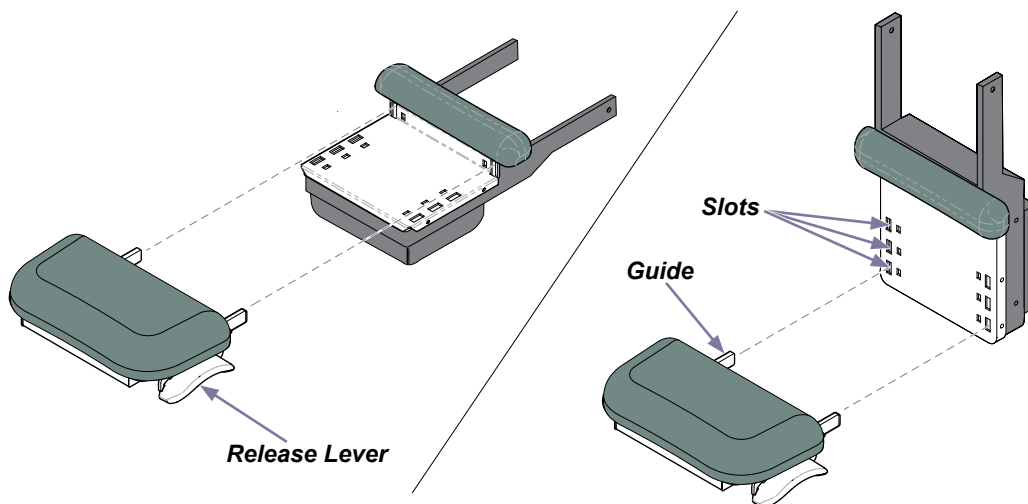

MA6847001

To install foot section in Knee / Chest Position...

- A) Lower the foot section.
- B) Insert foot section guides into desired slots.
- C) Be sure foot section is securely engaged before using.

Rotational Base **(optional)**


To rotate the table base...

- A) *Press and release the brake pedal.*
- B) *Rotate the table to desired position.*

To lock in position...

- A) *Press and release the brake pedal.*

Note: The base will automatically lock in position after two minutes.

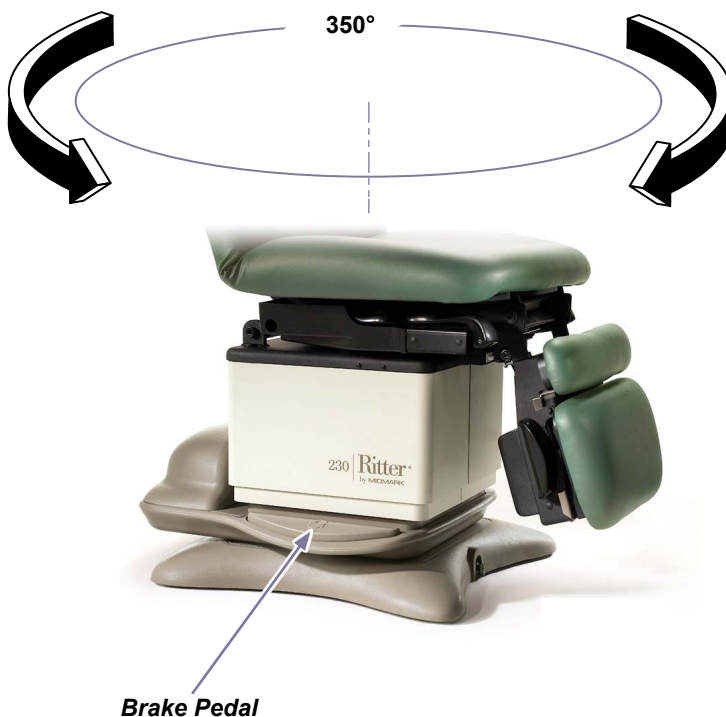Maintenance

Cleaning

Upholstery

Equipment Alert

The upholstery is resistant to most medicinal-type stains, but may be damaged by solvents and dyes. Immediately remove any fluids spilled on the upholstery.

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted / Plastic Surface

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Preventative Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your table every six months.

Calling for Service

[Product serial number is required]

Contact your Midmark dealer, or log onto www.midmark.com to locate your nearest service provider.

To contact Midmark directly:


1-800-Midmark (1-800-643-6275) or 937-526-3662

8:00 am until 5:00 pm. Monday through Friday (EST) [excluding standard U.S. holidays]

Troubleshooting

No Foot Up Function

Function	Status	Problem	Correction
Foot Up		No movement ("beeps")	Push treatment pan in.
All other functions		OK	-

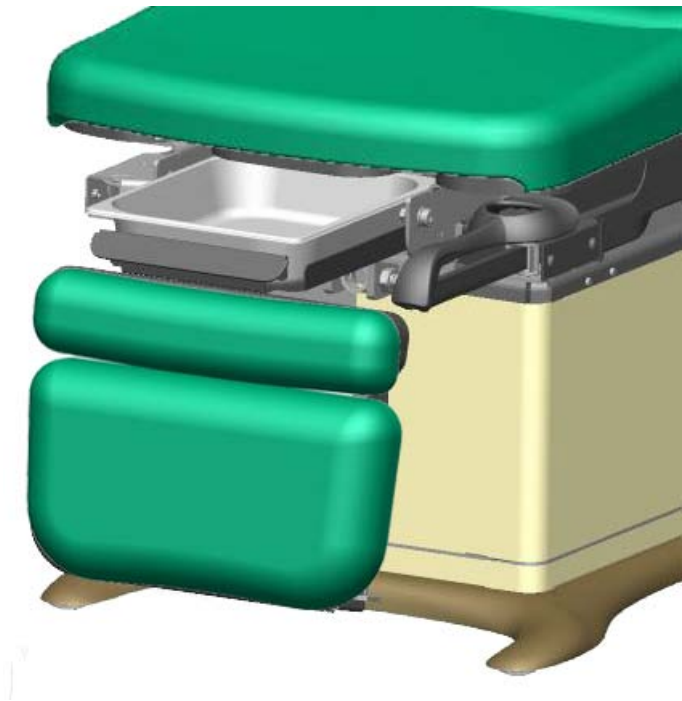Specifications

Weights, Dimensions, Electrical Specifications

Patient Weight (<i>maximum</i>):	450 lbs (204.1 kg)
Paper Roll (<i>maximum size</i>):	18 in. long x 3.5 in. diameter (45.7 x 8.9 cm)
Range of Motion & Dimensions:	[See <i>Range of Motion & Dimensions</i> page]
Weight of Table: 230 (w/standard base) w/packaging & skid 230 (w/rotational base) w/packaging & skid	455 lbs (206.4 kg) 510 lbs (231.3 kg) 540 lbs (244.9 kg) 600 lbs (271.1 kg)
Power Cord:	8 ft. (243.8 cm) long
Electrical Requirements:	[See <i>Compliance Charts</i>]
Foot Control Voltage:	10 VAC, SELV (Safety Extra Low Voltage)
Duplex Receptacle (optional) maximum load:	115 VAC, 3A, 50/60 Hz
Fuses: <i>Located at power cord inlet:</i> <i>F1 & F2</i>	6.3 A, 250V, Type T, 5 x 20 mm
Duty Cycle (motor run time):	Intermittent Operation [30 seconds ON / 5 minutes OFF]
Classifications:	Class 1, Type B Applied Part
Protection against ingress of fluids: <i>Foot Control <u>only</u>:</i>	Ordinary Equipment IPX1
Regulatory Compliance:	[See <i>Compliance Chart</i>]

WARNING

Equipment not suitable for use in the presence of a flammable anesthetic mixture.

Model Identification / Compliance Chart

Model	Description	Serial Number Prefix	Complies To:				Electrical Ratings:		
			UL 60601-1	CAN/CSA 22.2, #601.1-M90	IEC 60601-1	EN 60601-1-2 (EMC)	VAC +/-10%	Amps	Cycles (Hz)
230-001	Ritter Universal Proc. Table (Non-Programmable)	V	X	X	X	X	Configurable 115 230	7 3.5	50 / 60
230-002	Ritter Universal Proc. Table (Non-Programmable) w/ Receptacle	V	X	X	X	X	115	10	50 / 60
230-003	Ritter Universal Proc. Table (Non-Programmable) w/ Rotational Base	V	X	X	X	X	Configurable 115 230	7.5 3.8	50 / 60

Range of Motion & Dimensions

Max.
Standard base: 40 in. (101.6 cm)
Rotational base: 43.5 in. (110.5 cm)

Min.
Standard base: 22.5 in. (57 cm)
Rotational base: 26 in. (66 cm)

Max.
 $30^{\circ} \pm 2^{\circ}$

Min.
 $0^{\circ} \pm 0.5^{\circ}$



Max.
 $85^{\circ} \pm 3^{\circ}$

Min.
 $0^{\circ} \pm 0.5^{\circ}$

Max.
 $0^{\circ} \pm 0.5^{\circ}$

Min.
 $90^{\circ} \pm 1^{\circ}$

Range of Motion & Dimensions - (continued)

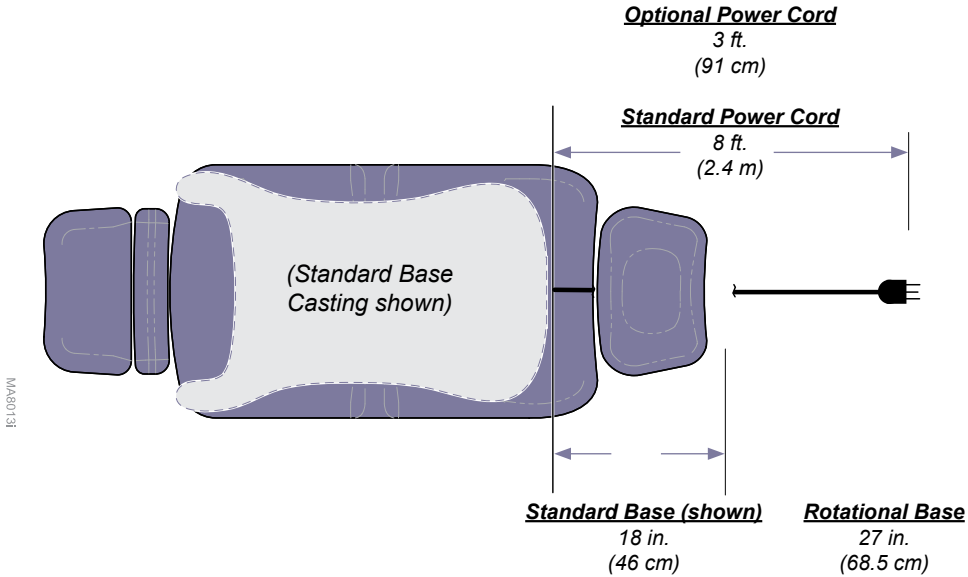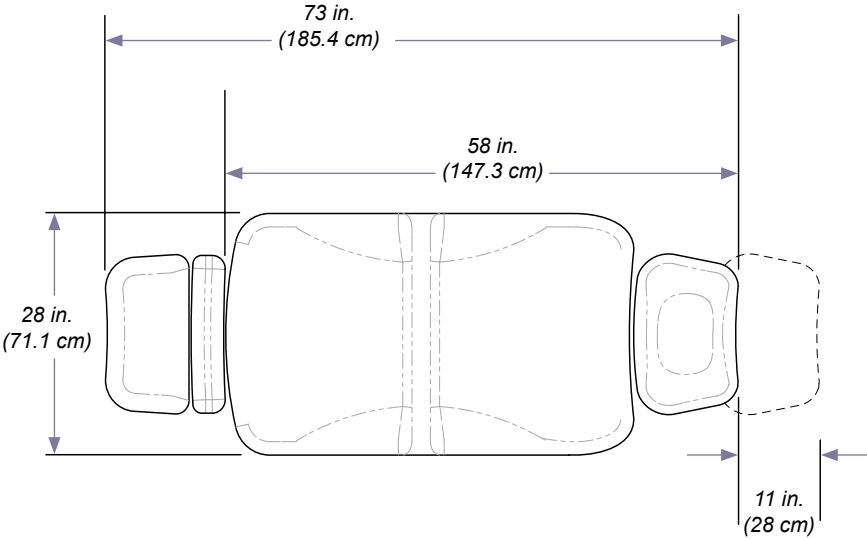Warranty Information

SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. Midmark's obligation under this warranty is limited to the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which, upon examination by Midmark, prove to be defective. This warranty extends only to the first retail purchaser of a product and is not transferable or assignable.

APPLICABLE WARRANTY PERIOD

The applicable warranty period, measured from the date of delivery to the original user, shall be one (1) year for all warranted products and components.

OBTAINING WARRANTY SERVICE

Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at www.midmark.com; by phone at 1-800-MIDMARK; by facsimile at 1-800-365-8631; or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380.

It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this warranty.

EXCLUSIONS

This warranty does not cover, and Midmark shall not be liable, for the following:

- (1) defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) products which are not installed, used, and properly cleaned and maintained as required in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product;
- (3) products considered to be of a consumable nature;
- (4) accessories or parts not manufactured by Midmark;
- (5) charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) costs and expenses of routine maintenance and cleaning; and
- (7) representations and warranties made by any person or entity other than Midmark.

Warranty Information (continued)

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER:

MIDMARK'S ONLY OBLIGATION UNDER THIS WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

NO AUTHORIZATION

No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

WARRANTY DISCLAIMER

THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS

No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

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