244

Barrier-Free™

Bariatric Power Treatment Table

Ritter®
by MIDMARK

[English](#)

[Español](#)

[Français](#)

User's Guide

Product Identification

(The information below is required when calling for service)

Model Number:

Serial Number:

Date of Purchase:

Name of Owner / Facility:

Name of Dealer:

Dealer's Phone Number:

Midmark Authorized Service Company:

MA678900p

Table of Contents

Important Information

Safety Symbols	4
Intended Use	5
Patient Positioning	5
Electromagnetic Interference	5
Transportation / Storage Conditions	5
Safety Instructions:	
<i>Hand Control Connection</i>	6

Operation

Power Switch	7
Table Up / Down	8
Back Up / Down	9
Tilt Up / Down	10
Paper Roll	11

Maintenance

Cleaning	12
Preventative Maintenance	12
Calling For Service	12

Specifications

Weights, Dimensions, Electrical Specifications	13
Regulatory Compliance Chart	14
Upholstered Top Dimensions	14
Range of Motion	15

Warranty Information

Limited Warranty	16
------------------------	----

Important Information

Safety Symbols

Caution

Indicates a potentially hazardous situation which could result in injury if not avoided.

Equipment Alert

Indicates a potentially hazardous situation which could result in equipment damage if not avoided.

Note

Amplifies a procedure, practice, or condition.

Consult User Guide
for important information.

Proper shipping orientation

Fragile

Type B, Applied Part

Protective earth ground

Keep dry

Maximum stacking height
(palletted units)

Fuse rating specification

Dangerous voltage / shock hazard

Duty Cycle (motor run time):
1 minute ON, 9 minutes OFF

[**ATTENTION:** Refer to *Regulatory Compliance Chart* to verify UL compliance for specific models]

This product is Listed by Underwriters Laboratories, Inc. Representative samples of this product have been evaluated by UL and meet the applicable U.S. safety requirements.

This product has been evaluated with respect to electric shock, fire, & mechanical hazards only, in accordance with UL60601-1, IEC60601-1, CAN/CSA C22.2, No. 601.1.

This product has been evaluated with respect to electrical shock, fire, & mechanical hazards only, in accordance with CAN/CSA C22.2, No. 601.1.

This component (or material) is Recognized by Underwriters Laboratories, Inc. Representative samples of this component have been evaluated by UL and meet the applicable U.S. safety requirements.

Intended Use

This product is intended to be used as a chair / table to provide positioning and support of patients during bariatric procedures conducted by qualified medical professionals.

Patient Positioning

The table is designed to accommodate the following examination positions:

- Prone
- Supine
- Lateral Sims

Caution

Never seat a patient on the back section of the table. Doing so may cause table to tip.

Electrical Requirements

Caution

Most exposed metal parts of the table are electrically grounded. When performing a cauterization or similar treatment, the patient must be insulated from the metal portions of the table by nonconductive material. Failure to do so could result in electric shock or burns to patient.

Electromagnetic Interference

This table is designed and built to minimize electromagnetic interference with other devices. However, if interference is noticed between another device and this table, remove interfering device from room and/or plug table into an isolated circuit.

Transportation / Storage Conditions

Ambient Temperature Range:-5°C to +38°C (+23°F to 100°F)

Relative Humidity 10% to 90% (non-condensing)

Atmospheric Pressure 500hPa to 1060hPa (0.49atm to 1.05atm)

Safety Instructions

Hand Control Connection

The hand control connection port is located on the table base at the head end of the table.


Equipment Alert

Never plug the hand control into any other outlet.
Doing so may result in damage to the equipment.

Caution

Be sure that all personnel and equipment are clear of the table before activating any function. Failure to do so could result in personal injury.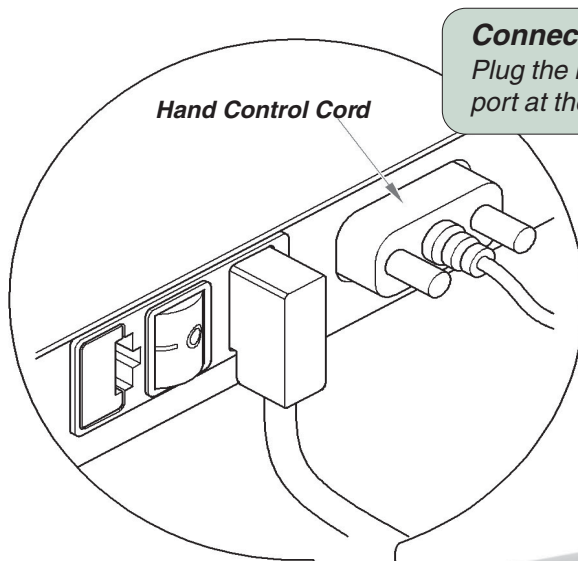

Connecting the hand control...

Plug the hand control cord into the port at the head end of the table.

Storing the hand control...

Hang the hand control on the magnetic bracket when not in use.

MA679700p

Operation

Power Switch


The power switch must be ON for the table functions to operate.

To turn the table ON...

Move the power switch to ON (I).

To turn the table OFF...

Move the power switch to OFF (O).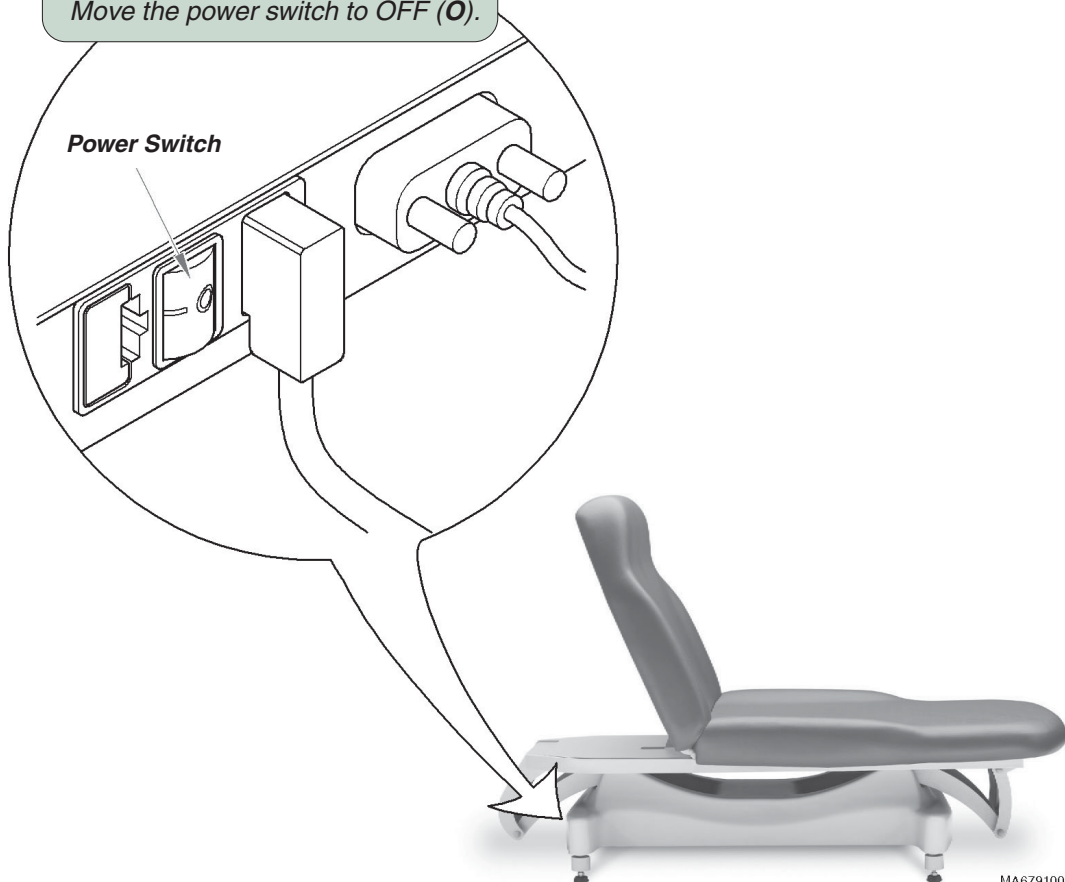

MA679100p

Table Up / Down

To move the table top UP or DOWN...
Press and hold the appropriate button.


Note: Movement automatically stops at the maximum / minimum height.

[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].

Max.
34 in.
(86.4 cm)

Min.
18 in.
(45.7 cm)
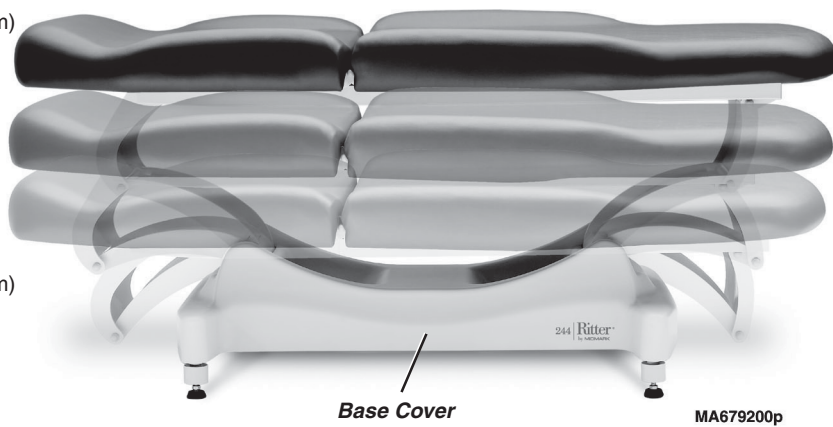


Back Up / Down

To move the Back section UP or DOWN...
Press and hold the appropriate button.

Note: Movement automatically stops at the maximum / minimum angle.

[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].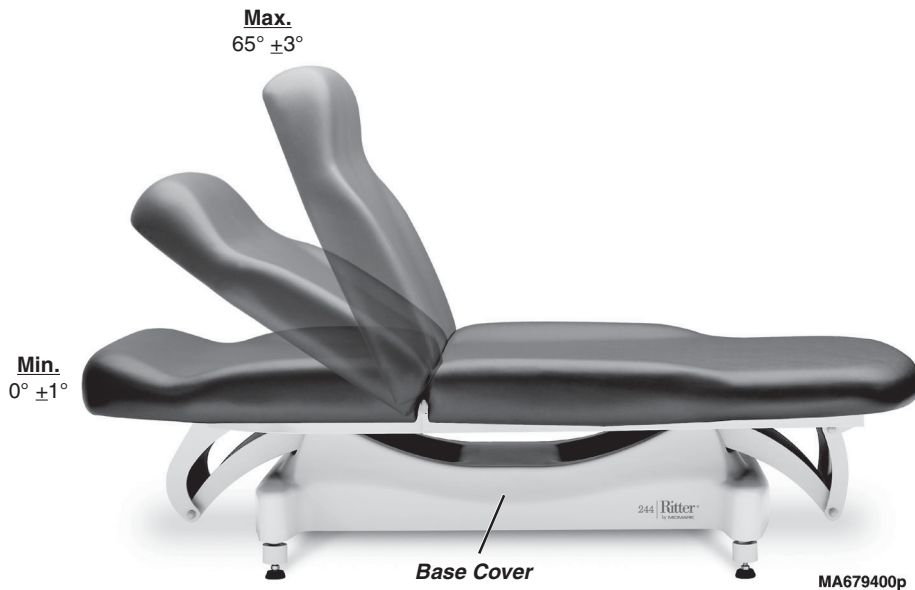
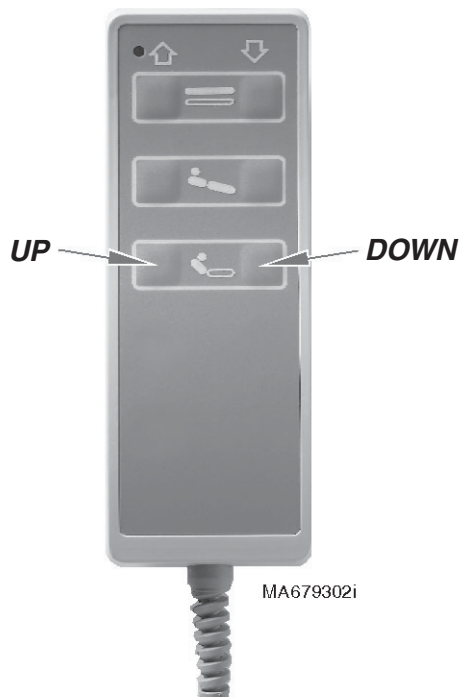


Tilt Up / Down

To Tilt the table top UP or DOWN...
Press and hold the appropriate button.

Note: Movement automatically stops at the maximum / minimum angle.

[The table is equipped with safety switches to stop all movement if any pressure is applied to the base cover].

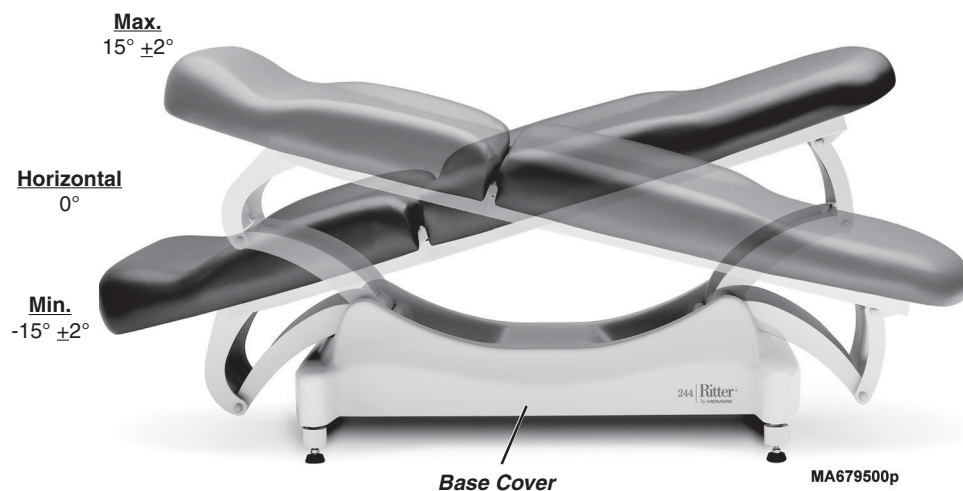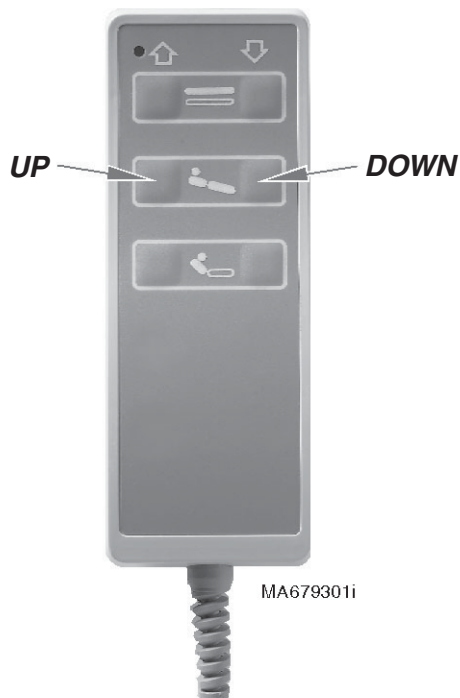
Paper Roll

Paper Roll Size (max.) 24 in. wide x 6 in. diameter
(61 cm x 15.2 cm)

Warning

To avoid injury, do not put head or extremities under the paper tear strap.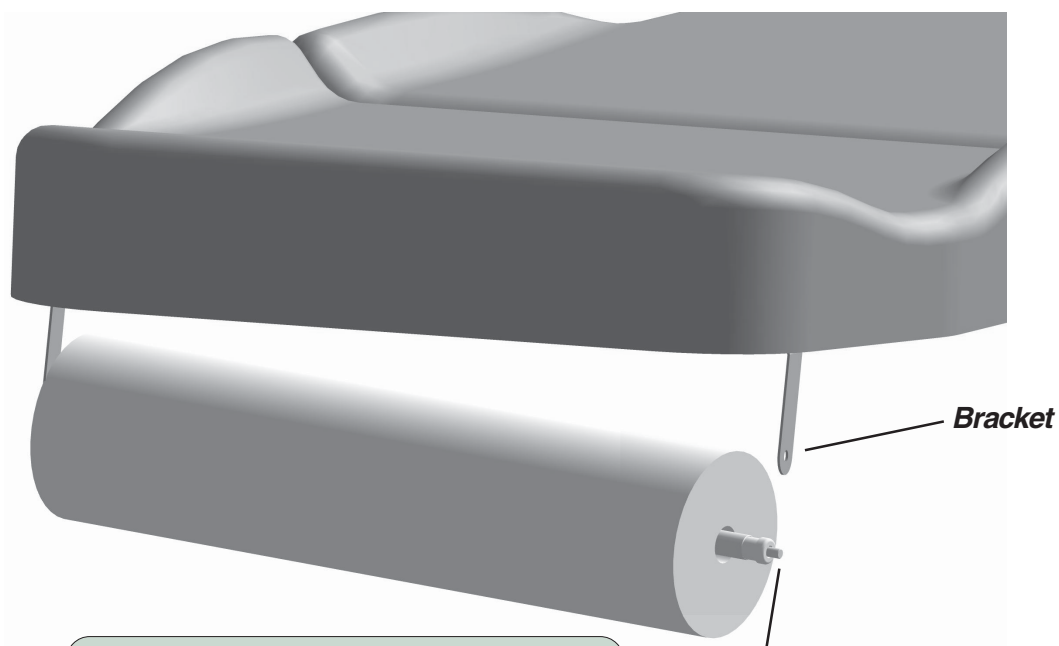

To install the paper roll...

- A. Install paper roll, onto dowel rod.
- B. Insert one end of dowel rod into bracket.
- C. Press button on other end of dowel rod, then insert rod into other bracket.

Cleaning

Upholstery

Wash your upholstery weekly with a mild liquid soap and water mixture, rinse with clear water and dry completely to remove disinfection cleaner build-up.

Disinfect your upholstery with a solution of standard bleach and water mixed 1 in 10 (10%) or chlorine based cleaners. Follow this with a clear water rinse and thorough drying of material. See current CDC Guideline for Disinfection & Sterilization in Healthcare Facilities.

To minimize damage caused by disinfectant cleaner residue build-up, do not allow disinfectants to pool on the upholstery surface. Once the approved contact time has been obtained, remove and dry excess liquid remaining on the surface.

Detailed care and maintenance instructions are included with your product. This information is also available on www.midmark.com in the Technical library under the User Information tab for your product.

Painted Metal / Plastic Surfaces

Clean the painted metal and plastic surfaces weekly using a clean soft cloth, and mild cleaner. Periodic applications of common furniture wax will ease cleaning, and maintain the finished luster of the surfaces.

Preventive Maintenance

Periodically inspect the following areas:

- Power cord should be free of cuts or other visible damage.
- All fasteners should be present and tightened securely.
- All mechanical functions should operate properly.

Have a qualified service technician inspect your table every six months.

Calling For Service

Note

Model / Serial Number information is required when calling for service.

Contact your Midmark Dealer, or log onto www.midmark.com to locate your nearest service provider. To contact Midmark directly:

1-800-Midmark (1-800-643-6275) or 937-526-3662
8:00 am until 5:00 pm. Monday through Friday (EST)
[excluding standard U.S. holidays]

Specifications

Weights, Dimensions, Electrical Specifications

Maximum Patient Weight: 850 lbs (385 kg)

w/Caster Base Accessory 650 lbs (295 kg)

Paper Roll (maximum size): (See *Paper Roll* page)

**Range of Motion &
Dimensions** (See *Range of Motion & Dimensions* page)

Weight of Table: 340 lbs (154 kg)

Power Cord: 8 ft. (243.8 cm) long

Fuses:

115 VAC models:

located in IEC inlet (2) 6.3A, 250V, Slo-Blo, 5 x 20 mm

Duty Cycle (*motor run time*): Intermittent Operation
[1 minute ON / 9 minutes OFF]

Classifications: Class 1, Type B Applied Part,
Ordinary Equipment, Intermittent Operation

Electrical Requirements: [See *Regulatory Compliance Charts*]


*Equipment not suitable for use in the presence of a flammable anesthetic mixture with air,
or with oxygen, or nitrous oxide.*

Regulatory Compliance Chart

Fire Code Ratings: All upholstery complies with California Bureau of Home Furnishing Technical Bulletin 117.

Model	Description	Serial Number Prefix	Complies To:					Electrical Ratings:		
			UL 60601-1	CAN/CSA 22.2, #601.1-M90	EN 60601-1-2 (EMC)	EN 60601-1	CE	VAC +/- 10%	Amps	Cycles (Hz)
244-001	Bariatric Treatment Table	n/a	X	X				120	2.5	60

Upholstered Top Dimensions

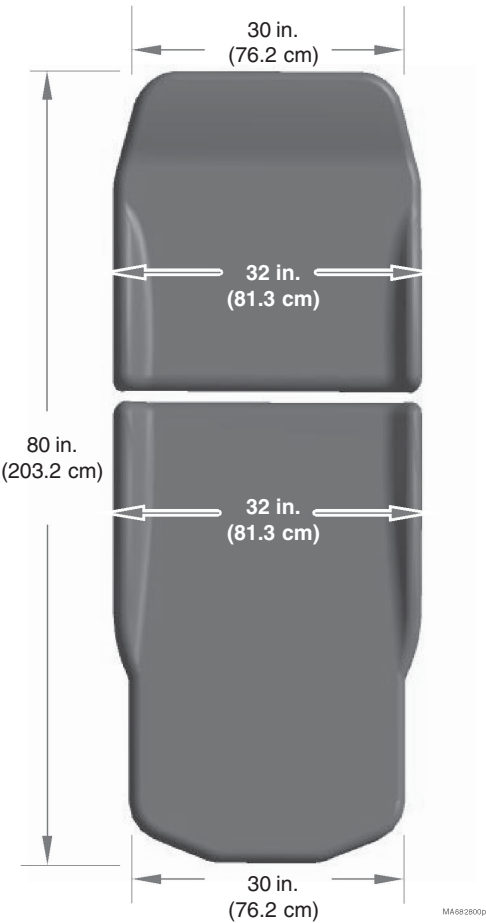Range of Motion

Max.
34 in.
(86.4 cm)

Min.
18 in.
(45.7 cm)

MA679200p

Max.
 $65^{\circ} \pm 3^{\circ}$

Min.
 $0^{\circ} \pm 1^{\circ}$

MA679400p

Max.
 $15^{\circ} \pm 2^{\circ}$

Horizontal
 0°

Min.
 $-15^{\circ} \pm 2^{\circ}$

MA679500p

Warranty Information

SCOPE OF WARRANTY

Midmark Corporation ("Midmark") warrants to the original retail purchaser that it will repair or replace components of the domestic and international medical products manufactured by Midmark (except for components not warranted under "Exclusions") that are defective in material or workmanship under normal use and service. Midmark's obligation under this warranty is limited to the repair or replacement, at Midmark's option, of the applicable components. This limited warranty shall only apply to defects that are reported to Midmark within the applicable warranty period and which, upon examination by Midmark, prove to be defective. This warranty extends only to the first retail purchaser of a product and is not transferable or assignable.

APPLICABLE WARRANTY PERIOD

The applicable warranty period, measured from the date of delivery to the original user, shall be one (1) year for all warranted products and components.

(A) Ultrasonic cleaner products are guaranteed for a period of two (2) years after ship date when used in accordance with manufacturer's instructions.

(B) Replacement parts and accessories carry a ninety (90) day warranty.

OBTAINING WARRANTY SERVICE

Warranty service must be obtained through either Midmark or an authorized dealer in the Midmark product line for which warranty service is requested. Midmark may be contacted for warranty service inquiries or issues via email at www.midmark.com; by phone at 1-800-MIDMARK; by facsimile at 1-800-365-8631; or by mail to Midmark Corporation, 60 Vista Drive, Versailles, Ohio 45380.

It is the retail purchaser's obligation to arrange for delivery of a product to Midmark or one of its authorized dealers for warranty service, which delivery shall be at retail purchaser's expense. It is also the retail purchaser's obligation to comply with the warranty service instructions provided either by Midmark or its authorized dealer. The retail purchaser must provide Midmark with completed warranty registration information within thirty (30) days after purchase in order to obtain the benefits of this warranty.

Warranty Information (continued)

EXCLUSIONS

This warranty does not cover, and Midmark shall not be liable, for the following:

- (1) defects, damage or other conditions caused, in whole or in part, by misuse, abuse, negligence, alteration, accident, freight damage, tampering or failure to seek and obtain repair or replacement in a timely manner;
- (2) products which are not installed, used, and properly cleaned and maintained as required in the Midmark "Installation" and/or "Installation/Operation Manual" for the applicable product;
- (3) products considered to be of a consumable nature;
- (4) accessories or parts not manufactured by Midmark;
- (5) charges by anyone for adjustments, repairs, replacement parts, installation or other work performed upon or in connection with such products which are not expressly authorized in writing in advance by Midmark;
- (6) costs and expenses of routine maintenance and cleaning; and
- (7) representations and warranties made by any person or entity other than Midmark.

EXCLUSIVE REMEDY; CONSEQUENTIAL DAMAGES DISCLAIMER:

MIDMARK'S ONLY OBLIGATION UNDER THIS WARRANTY IS THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS. MIDMARK SHALL NOT BE LIABLE FOR AND HEREBY DISCLAIMS ANY DIRECT, SPECIAL, INDIRECT, INCIDENTAL, EXEMPLARY OR CONSEQUENTIAL DAMAGES OR DELAYS, INCLUDING, BUT NOT LIMITED TO, DAMAGES FOR LOSS OF PROFITS OR INCOME, LOSS OF USE, DOWNTIME, COVER AND EMPLOYEE OR INDEPENDENT CONTRACTOR WAGES, PAYMENTS AND BENEFITS.

NO AUTHORIZATION

No person or firm is authorized to create or approve for Midmark any other obligation or liability in connection with the products.

WARRANTY DISCLAIMER

THIS WARRANTY IS MIDMARK'S ONLY WARRANTY AND IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED. MIDMARK MAKES NO IMPLIED WARRANTIES OF ANY KIND INCLUDING ANY IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE. THIS WARRANTY IS LIMITED TO THE REPAIR OR REPLACEMENT OF DEFECTIVE PARTS.

STATUTE OF LIMITATIONS

No action may be brought against Midmark for breach of this limited warranty, an implied warranty, if any, or for any other claim arising out of or relating to the products, more than ninety (90) days following expiration of the limited warranty period.

Midmark Corporation
60 Vista Drive
Versailles, OH 45380-0286
Phone: 937-526-3662
Fax: 937-526-5542
www.midmark.com

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