



MJM INTERNATIONAL CORPORATION

HOW TO ATTACH CHEST AND SHOULDER HARNESS

IDENTIFY COMPONENTS TO START ASSEMBLY
PARTS INCLUDED



HARNESS (FRONT VIEW-BACK VIEW).

STEP 1 – REFERENCE PHOTO



PLACE HARNESS OVER BACK OF CHAIR.

STEP 2



BEFORE THE PERSON IS SEATED UNSNAP SHOULDER STRAP/UNSNAP WAIST STRAP.

STEP 3



AFTER THE PERSON IS SEATED SNAP THE SHOULDER STRAP/WAIST STRAP.

STEP 4



ADJUST WAIST STRAP AS NEEDED.
ADJUST SHOULDER STRAPS AS NEEDED.

STEP 5



PICTURE REFERENCE CHEST SHOULDER HARNESS COMPLETED

NOTE: HARNESS MUST BE CHECKED AND PROPERLY SECURED PRIOR TO USING.

WARNING!!!! NOT SECURING HARNESS CAN CREATE A SERIOUS SAFETY HAZARD.