



Nova Transport Chairs

300 SERIES

(Model #s 307/309/319/327/329/330/332/348/319/352)



Features

- Lightweight Aluminum Frame
- Includes seatbelt.
- Padded upholstery & armrests.
- Removable Aluminum Footrests
- Locking Rear Wheels.
- Non-skid Footplates & Heel Loops.

WEIGHT CAPACITY 300 lbs. – 400 lbs.

OWNER'S MANUAL

OWNER

YOU MUST READ THIS MANUAL BEFORE USING THIS PRODUCT

DEALER

YOU MUST SUPPLY THE OWNER WITH THIS MANUAL

WARNINGS

DO NOT Use this product without first reading and understanding this manual.

DO NOT Use this equipment without first reading and understanding Nova's terms.

If you are unable to understand the, CAUTIONS, TERMS AND CONDITIONS, AND INSTRUCTIONS, then contact a healthcare professional, dealer, lawyer, or a qualified technician before attempting to use this product - OTHERWISE INJURY OR DAMAGE MAY RESULT.

By using or continuing to use this product, YOU ARE AGREEING TO THE NOVA TERMS AND CONDITIONS detail at www.NovaMedicalProducts.com.

For questions regarding this manual or the product, please contact us below.

Distribution Center
Nova Ortho-Med, Inc.
8730 West 50th Street
McCook, IL 60525

Corporate Office
Nova Ortho-Med, Inc.
1470 Beachey Place
Carson, CA. 90746

1-800-557-6682

www.NovaMedicalProducts.com



WARNING

Failure to follow any or all safety instructions may result in serious injury or death.

- DO NOT ADJUST, MAINTAIN, OR OPERATE THIS TRANSPORT CHAIR WITHOUT READING, UNDERSTANDING, AND FOLLOWING, THE PROPER INSTRUCTIONS INCLUDED WITH THIS PRODUCT

- User weight not to exceed 300 lbs. (Model #s 307/309/319/327/329/330/349/352)
- User weight not to exceed 250 lbs. (Model # 348)
- User weight not to exceed 400 lbs. (Model # 332)
- ALWAYS operate the chair with the back support in the upright locked position.
- ALWAYS secure the user to the chair using the seat belt.
- ALWAYS ensure that all grips and textured surfaces are secure before pushing, pulling, or placing weight on such surfaces.
- ALWAYS perform the initial product inspection and all follow-up inspections before using the product.
- ALWAYS engage the wheel locks upon entering or leaving the transport chair.
- ALWAYS test the push handles to make sure that both push pins are securely in place before sitting in the chair
- DO NOT operate this transport chair on streets or roadways.
- DO NOT stand on or put pressure on footrests when entering or exiting the transport chair.
- DO NOT use the wheel locks to slow your descent down an incline.
- DO NOT change directions when going down an incline. This may cause instability.
- DO NOT scoot on or operate the chair without an assistant maintaining a secure grip from behind the chair.
- DO NOT sit on chair unless the product is fully open and in its normal usable position.
- DO NOT use ramps or operate the product on inclines greater than 9 degrees of slope.
- DO NOT operate on moving walkways, escalators, roads, streets, or highways (Use extreme caution when operating product in crowded areas or outdoors).
- DO NOT operate the chair in bad weather or when ground conditions may impair traction, such as ground with water, oil, ice, gravel, or any other slippery or unstable substance.
- DO NOT attempt to cross curbs, bumps, trenches, or other uneven surfaces. Doing so may tip over the product. Operate product only on smooth well-maintained surfaces.
- DO NOT attempt to reach objects while using product as this requires you to move your weight, and can cause the product to tip over.
- DO NOT reach for objects, as this requires you to move your weight, and can cause the product to tip over.
- DO NOT pick up objects from the floor, as this requires you to move your weight, and can cause the product to tip over.
- DO NOT tilt the wheelchair while occupied.
- DO NOT use the wheel locks as brakes, nor attempt to stop a moving product by engaging the wheel locks.
- DO NOT attempt to lift the product by pulling on any removable part or surface.
- DO NOT stand on the product, the product frame, or any other part of the product.
- DO NOT use the footrests as a platform. When attempting to enter or exit the product, ensure that the footrests are in the upward, unusable position.
- DO NOT carry heavy weight while occupying the product. The product is not designed for additional weight, and is not balanced to accommodate additional weight.
- DO NOT lift weights or exercise while occupying the product.
- This transport chair has not been approved as a seat surface to be used within a vehicle.
- Transport chair must be on a level surface when transferring user to or from seat.
- Footrests should be no less than 2 1/2 inches from the floor/surface.

OPERATING INSTRUCTIONS

Assembly

To Unfold

1. Remove chair from the box and place it on the ground. Remove any cardboard protection and the plastic from the wheels.
2. Bring the folding push handles upward fully to enlarge and lock the push pins into the frame of the chair.
3. Place both hands on the upholstery just to the inside of the seat tubes and push down and out until the seat is fully opened. Your fingers could be pinched, therefore, be sure both hands and all fingers are flat on the upholstery. Do not allow fingers to grab or reach the black tubes that secure the seat. Continue to push down and outward until tubes engage into the seat supports. (Warning: failure to upholster could lead to possible hand or finger injury. Please call Nova customer service if you have any questions.)

To Fold

1. Pull upward on the middle of the seat from the front and rear until the chair is completely folded.
2. Release and fold down the back/push handles by pushing down on the release paddle and allowing the handles to come down into a folded position.

To Attach the Footrests

1. First determine the left and right side by folding down the footplate and placing them in front of the seat. The footplates should be facing each other and directly in front of the seat.
2. Place the attachment plates with holes above the pegs and slowly lower the holes onto both pegs.
3. Swivel the footrests inward until the silver release lever engages and locks the footrest into the forward position.

To Remove the Footrests

1. Push the silver release lever inward to release the footrests from the forward position.
2. Swivel outward and lift the footrests from the footrest pegs.

To Adjust the Footrest Height

For Models 307/309/319:

1. Locate tool wrench included with the footrests.
2. Turn wheels so they are in the "wheeled" position (as pictured).
3. Use the tool wrench to slightly loosen the silver hex screw located on the bottom of the footrests by turning the screw left.
4. Pull on the footplate to desired length. Footrests should be a minimum of 2 1/2" from the ground to ensure proper clearance.
5. Re-tighten the hex screw.
6. Repeat on other side.

For Models 327/329/330/332/348/349/352:

1. Remove height adjustment knob by turning counter clockwise.
2. Adjust the inner tube to desired height and line up hole on inner telescoping tube with outer hole.
3. Screw adjustment knob back into the adjustment leg and tighten.

To Use Hand Brakes (Models 330/332/352)

The hand brake is to be used for speed control only. Squeeze up on the hand brake to control the speed. **Use the wheel brake when entering or exiting the chair.**

Engage the Park Brake

Push down on the black ball shaped knob located at the rear of the hand brake handles. The handle will lock into a downward position when properly engaged. Pull up on the hand brakes to release the brakes.

How to Adjust the Brakes

Note: The braking system on your new Transport Chair with Hand Brakes has been preset from the factory. If you need to adjust the tension of your brakes, you can adjust them by the following:

Hand Brake is Too Loose.

Loosen the tension adjustment screw locking nut and turn the screw one full turn clockwise (as viewed from standing at the back of the transport chair). If this does not correct the adjustment, repeat one full turn at a time until you have adjusted your brakes. Then make sure the locking nut is spun tight against the housing to prevent the screw from loosening.

Hand brake is Too Tight.

Same as the procedure above except turn the screw counter clockwise.

User Instructions

To Sit Down

1. Engage the rear wheel locks so that the wheels by pushing down on the brake handle until the brake shoe presses firmly on the wheel and prevents the chair from rolling.
2. Flip the footrests into an upright position so that the user can stand between the footrests with the back of their legs touching the seat upholstery.
3. From behind the chair assist the user by slowly lowering them into the chair.

To Rise from Seated Position

1. Engage the rear wheel locks so that the wheels by pushing down on the brake handle until the brake shoe presses firmly on the wheel and prevents the chair from rolling.
2. Flip the both footrests into an upright position.
3. Place both feet firmly on the ground and rise slowly from the chair with the assistance of a care giver.

To Transfer from Seated Position (Model #s 349/352/332)

1. Place the chair along side of the desired seating device.
2. Engage the rear wheel locks so that the wheels by pushing down on the brake handle until the brake shoe presses firmly on the wheel and prevents the chair from rolling.
3. Remove or flip back the arm on the side of the transfer by pressing on the plastic covers located on the front and rear of the arm.
4. Release the footrests and swing them to the outer sides of the chair frame or flip both footplates upward to a vertical position.
5. Carefully transfer from the chair to desired seating area with the assistance of a caregiver.

General Care

Wipe all metal parts at least once a week using a soft cloth. Dry chair immediately if it is exposed to moisture. Abrasive cleaners or materials will scratch the surface of the chair.

Tires

Minimal maintenance is required for rubber tires. Wipe with a damp cloth occasionally. If tires become cracked or worn, replace immediately.

Front Casters

Inspect casters periodically and lubricate them when necessary. To lubricate, remove the fork and grease the caster bearings. Remove the wheel from the fork and grease the wheel bearings. The wheel and fork can be reassembled by tightening the nuts until there is no play remaining, but the wheel rolls freely. The fork will revolve freely when adjusted correctly. The caster will utter if the nut is too loose or be difficult to steer if the nut is too tight.

Wheels

Inspect monthly for correct tightness. If the wheel seems to be loose or wobbly, release the lock nut and tighten the axle bolt. Lean chair to one side and spin adjusted wheel to check for correct adjustments.

Wheel Locks

Wheel locks should be checked for correct locking action before actual use. Pushing the handle completely forward activates the wheel locks. If the wheel locks are working correctly they will be embedded into the tire at least 1/8". To adjust wheel locks, loosen nut on carriage bolt. Slide clamp toward rear wheel until lock shoe is embedded at least 1/8" into rear tire when lock handle is engaged. Tighten nut to secure into position.

- **Do not attempt to repair the transport chair yourself.**
- **Call the store where you purchased the transport chair. Most dealers do repairs and will re-order parts for you.**
- **If your dealer is unable to help you, please call Nova at (800) 557-6682.**

SERVICE

All service must be performed while the wheelchair is unoccupied.

Minor adjustments may be attempted by an able bodied individual. All minor adjustments that one should attempt are detailed in the manual. Do not attempt any adjustment or repair without reading and understanding the instructions in this manual. Do not attempt any adjustment or repair that is not detailed in this manual. Any adjustment or repair not detailed in this manual should only be performed by a dealer or licensed technician.

Individuals with muscular or balance problems should not attempt to make adjustments to the product, and are advised to seek assistance.

After any adjustment, and before using the product, ensure that all attaching hardware, including without limitation all grips, screws, nuts, bolts, clamps, fasteners, and parts are tight and secure.

DO NOT use parts, accessories, or adapters other than those manufactured by Nova Ortho Med., Inc, which are designed specifically for this product.

Warranty Information

Nova stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

Nova extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. Nova warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame: Limited Lifetime Warranty

Hand Brake: 5 years Limited Warranty

(Plastic Handle & Cable)

Non-Metal/Metal Parts: 1 Year Limited Warranty

(Wheels, Wheel Brakes, Hand Grips, Rubber Tips, Cable Covers, All other Hardware & Plastic Parts)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your Nova product. In the event that you do not receive satisfactory service, please call us directly at the number below. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. DO NOT return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

Customer Help Line: 1-800-557-6682

www.NovaMedicalProducts.com

P.O. Box 3039 • Gardena CA 90247