

The Caregivers' Repositioning Solution



The ErgoSheetTM is the only repositioning device designed by caregivers for caregivers. Constructed from a patented nylon material, the ErgoSheetTM assists caregivers in repositioning a patient in bed.

- Easily reposition a patient in bed
- Effortlessly turn a patient in bed
- Ergonomically inclined handles to assist in "No Lift" maneuvers

General

The ErgoSheet™ is an ultra-low friction ergonomically inclined device, which aid caregivers in moving or positioning patients while encouraging good posture without causing undue stress to either the patient or the caregiver.

The ErgoSheet™ was not designed to, and must not be used to lift patients. This repositioning device also encourages patients to help themselves whenever possible.

Important Prior To Usage

This brochure is only a guide, as alternative methods may be used for patient handling. We strongly recommend that your facility administrator or risk manager review and approve all protocols used for safe patient handling. Prior to using any equipment, be sure to check for signs of wear to ensure proper functionality. When using the ErgoSheet™ for the first time, practice with a fellow caregiver until you have a level of comfort with the product.

Cleaning Instructions

The ErgoSheet™ can be wiped clean using a mild agent or machine washed and dried per label instructions.

Important!

- Please ensure that the ErgoSheet™ is packed away or hung up after use in order to prevent people accidentally slipping on them.
- The ErgoSheet™ was not designed to be left directly under the patient. However, in the case where there is a need for long term patient positioning in bed it may be possible, based on a clinical evaluation and with proper supervision of the patient, using an interface such as a cotton sheet, to leave the product under the patient.
- Please check the ErgoSheet™ regularly and replace immediately if sheets show signs of wear.
- The ErgoSheet™ has not been designed to lift patients.

Visit us online at www.ergosafeusa.com to see our full line of ergonomically designed products.



ErgoSafe Products, LLC • 142 Firwood Drive, St. Louis, MO 63119
314-968-9066 • www.ergosafeusa.com