

# SLING INSTRUCTION SHEET

## WAVERLEY GLEN ROLLER BAND SLING SET


### GENERAL GUIDELINES

Under no circumstances should any Waverley Glen Lift System be put in the control of persons who have not received proper training in its operation. If operated improperly, this lift system can cause serious injury.


Although the lift system is designed to be operated by an attendant, it may be operated by an unattended individual, providing this person has been properly trained in the system's use. The safety of both the attendant and the individual depends upon strict adherence to the following guidelines:

1. All instructions pertaining to the operation of the lift system should be read prior to the system being put into use.
2. Do not, under any circumstances, exceed the system's standard working load.
3. Prior to using the sling, make a visual inspection for signs of wear. Look for fraying, cuts, or tears in the material of both the sling's body and straps.
4. The sling must be positioned properly under and around the individual, to ensure a balanced lift.
5. The lift system is not a toy; do not use it for unsafe practices.

In facilities when more than one staff member is responsible for operating the lift system, it is imperative that all such members be trained in its proper use. A training and orientation program should be established by the facility to acquaint new caregivers with the system.

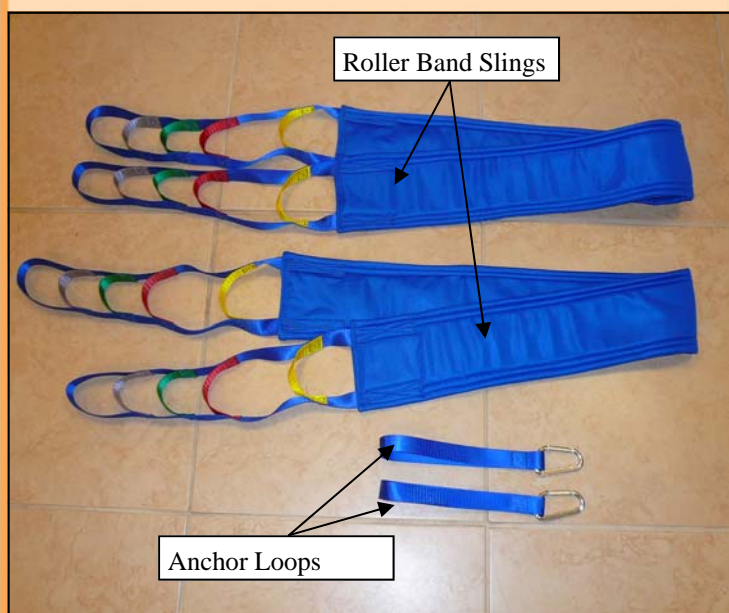

1) Locate an anchor point on the bed frame. Wrap the closed webbing anchor assembly around the anchor point as shown. Two anchor points are required on the same side of the bed. One located at the patient's lumbar area, the other behind the knees.

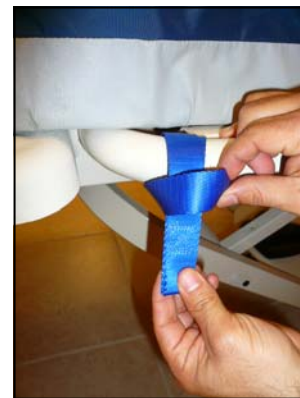

2) Feed one end of the anchor loop through the other to form a solid anchor point.



3) Hook the carabiner through the loop in the anchor loop as shown.

**WAVERLEY GLEN**  
A PRISM MEDICAL COMPANY

Waverley Glen Systems Ltd.

87 Sharer Road, Vaughan, ON Canada L4L 8Z3 | T 905.850.0093 | 1.800.265.0677 | [www.waverleyglen.com](http://www.waverleyglen.com)

# SLING INSTRUCTION SHEET

## WAVERLEY GLEN ROLLER BAND SLING



4) Slide the roller bands under the patient as shown. Patient lifting is not required to perform this function. Slide one band under the lumbar spine area and the other one under the knees.



5) Attach one side of each band to each anchor point using the carabiner as shown.



6) Move ceiling lift or floor lift in place and rotate the carry bar so it is perpendicular to the shoulders. Attach the loops to the hooks in the carry bar.



7) Depress the up button on the lift to easily roll the patient into the desired position.

### WASHING INSTRUCTIONS

The sling should be hand washed because the metal rings and plastic seat belt may damage the washing machine or vice versa. The sling should be hung to air dry. Warm wash with mild detergent is recommended.



**WAVERLEY GLEN**  
A PRISM MEDICAL COMPANY

Waverley Glen Systems Ltd.

87 Sharer Road, Vaughan, ON Canada L4L 8Z3 | T 905.850.0093 | 1.800.265.0677 | [www.waverleyglen.com](http://www.waverleyglen.com)